

**Adalimumab Levels  
Instructions for Laboratory**

1. Enter the test into Sunquest as MISAS-;ADALIMUMAB LEVEL
2. Collect either a Gold Top or a Red Top
3. Spin down and aliquot into 4 tubes.
4. 2 tubes will need a min. of 0.5 mL of serum and 2 tubes will need a min. 0.25 mL
5. Freeze samples.
6. Send as normal frozen to Edmonton
7. Please send Attention: Cara Shalapay / Send Out Bench / UAH

**Cara must receive all 4 aliquots as well as the signed ADA requisition to accompany the samples.**

If you have any questions, please contact:

Cara Shalapay

Laboratory Assistant II - Clinical Referred Out Testing/Client Response/Urinalysis

Alberta Health Services / University of Alberta Hospital, Room 4B3.04 Lab Medicine

Phone: (780)407-7463 Fax: (780)407-6267